

FACIAL PROTECTION VISOR

« Reflexes »
sheet

This visor can be used as a face shield to prevent liquid splashes (sprays, airborne particles) or dust.

USAGE

- Remove the protective film before use.
- The foam headband helps to reduce pressure.
- Does not fog when wearing the mask.
- Will fog if the nose touches the screen, in this case, change the setting by lowering the gripping bands towards the back of the neck.
- Fogging when there is a significant difference in temperature.

MAINTENANCE

- It is important to clean both sides as regularly as possible.
- It is recommended to wash this anti-spray protective screen with a microfiber cloth to avoid scratching the glass and hydroalcoholic gel.
- Be careful to disinfect the entire visor, including the straps. Remember not to leave the soiled cloths lying around once the protective screen has been cleaned.
- Wash your hands once finished.

INFORMATION

- The transparent protective visors do not replace the masks, it is a complement because this transparent screen protects above all the eyes from possible projections and allows the people who wear glasses not to contaminate them.
- This protective equipment limits the risk of particles being projected onto the masks and therefore allows them to be kept longer.